

Reliant™

Stent Graft Balloon Catheter

FEATURES

Expand Possibilities

A single-solution balloon catheter for your stent graft procedure needs

Clinical uses include:

- Abdominal and thoracic use
- Endograft modeling
- Endoleak sealing support

Widest Range of Balloon Inflation Diameters

BALLOON INFLATION TABLE

46 mm Balloon	
Diameter (mm)	MI (cc)
10	3
20	9
30	19
40	41
46 [†]	60

CAUTION: This table is only a guide. Balloon expansion should be carefully monitored under fluoroscopy. Do not exceed maximum inflation diameter (46 mm). Rupture of balloon may occur.

PRODUCT INFORMATION

RELIANT™ STENT GRAFT BALLOON CATHETER[‡]

Product Code	Inflation Diameter (mm)	Shaft Size (Fr)	Usable Length (cm)	Sheath Compatibility (Fr)
AB46	10–46	8	100	12

Please reference appropriate product *Instructions for Use* for a more detailed list of indications, warnings, precautions and potential adverse events.

[†] Maximum inflation diameter.

[‡] Does not contain latex.



MULTIPLE PURPOSES, SINGLE SOLUTION

